


INSTRUCTIONS FOR USE

- 1 The drug is introduced through the cone of the syringe body.
- 2 The injection needle is screwed on to the Luer-Lock top.
- 3 If the introduced volume is less than 5 ml, the plunger is pushed by the positioner until the air in the syringe has gone.
- 4 The prepared syringe body is put into the pole end.
- 5 The holder of the pole is pushed over the needle.
- 6 The gas cartridge is pushed vertically on the valve at the other end of the pole. By short pressure, some gas is let in.
- 7 After use, the plunger is pushed back in its starting position by using the hypodermic syringe with the connecting piece.
- 8 When using the extension element, the pole syringe is put into its opening and fixed by screwing the milled nut.
- 9 Each extension element can be screwed one into the other by removing the milled nut of the second element.

BEFORE NEXT USE, THE REMAINING GAS IN THE POLE MUST BE LET OUT BY PUSHING THE EMPTY SYRINGE BODY AGAINST THE VALVE INSIDE THE POLE

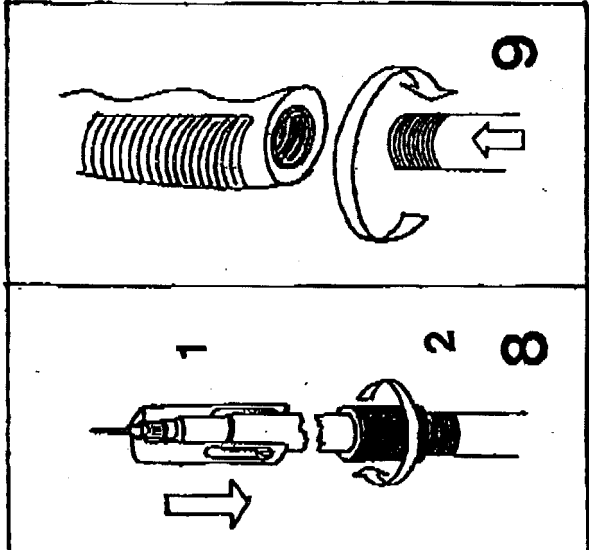

Dist-Inject