

ACES Animal Stretcher Assembly Instructions

Parts:

Stretcher Frame x1

Stretcher Material x1

Phillips Head screws x2

Wheels x2

Wheel Bar x1

Strap x1

Large Washers x4

Wing Nuts x2

Split Pins x2

Eyebolt Assembly (Eyebolt, Wing nut, Washer) x2

Steps:

- Kit will come with stretcher material already attached to frame.

Fig. 1

- Attach frame tubing to create full stretcher.
- Insert Phillips head screws into holes in middle of frame to anchor frame assembly. - optional
- Pull stretcher material taut, using the velcro strips on both ends to anchor

Fig. 2a & 2b

- Insert wheel bar into machined holes on the end of stretcher (the wheel bar pegs will only fit on one side of stretcher) and attach with wing nuts

Fig. 3

- Attach wheels (large washer – wheel – large washer) to the sides of the wheel bar, use the split pins to keep the wheels in place

Fig. 4a & 4b

- Attach eyebolt assembly on opposite side of stretcher; make sure that the eye of the bolt is on the same side as the restraint flap of the stretcher
- Strap will clip to the eyebolts allowing a single person to pull the loaded stretcher.
- Adjust strap to desired length.

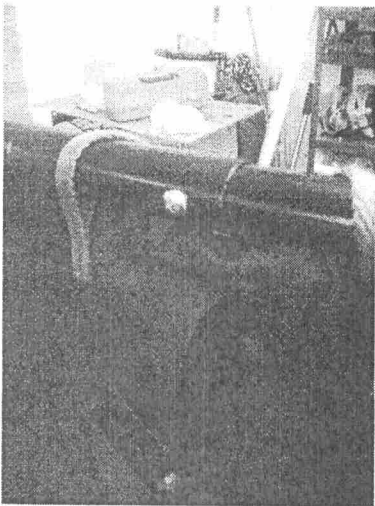


Fig. 1

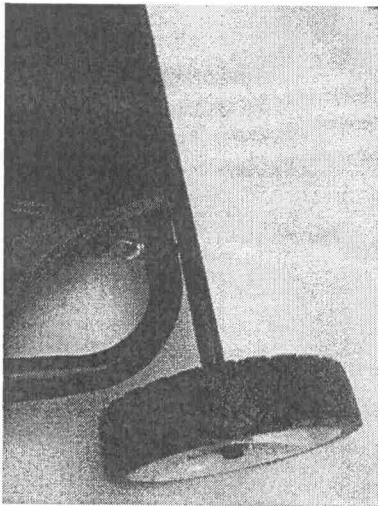


Fig. 2a

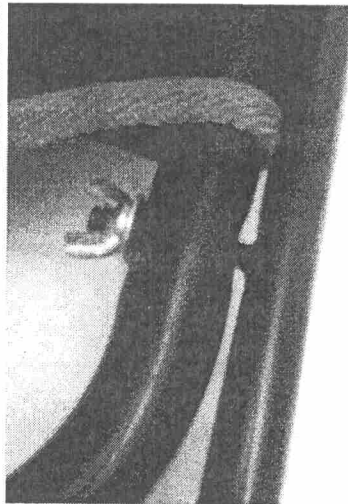


Fig. 2b

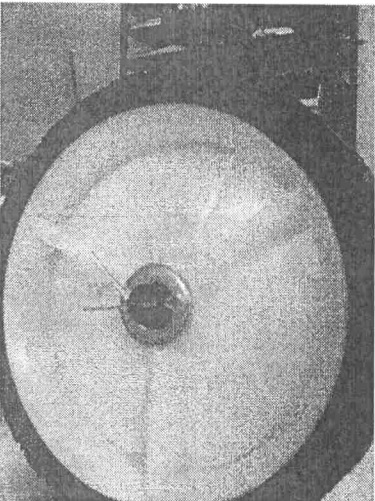


Fig. 3

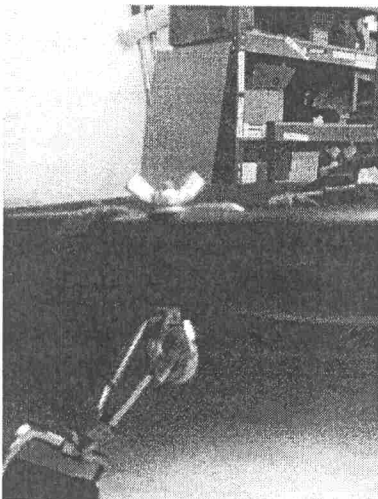


Fig. 4a



Fig. 4b